MANUAL DE USUARIO / INSTRUCTIONS FOR USE

REF 2190

Sucesores de Manuel García S.L.
CIF B12321162
Carretera de Castellnovo, Km 0, 12400 Segorbe (Castellón), Spain
Tel.: +34 964 71 03 22 - e-mail: comercial@garcia1880.com
www.garcia1880.com

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SILLA DE DUCHA DE ALUMINIO

REF 2190

NORMATIVA

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REF 2190

NORMATIVE

This product is a Class I medical device and complies with Regulation (EU) 2017/745 of the European Parliament and of the Council of April 5, 2017 on Medical Devices.

This product should not be disposed of in the waste container. There are specific collection points where you can deposit them. For the disposal of the container and the product, strictly comply with the legal regulations of your community.

INTENDED USE

Support device for washing and bathing. It is a technical aid that allows people with reduced mobility or cannot make conventional use of the shower, to remain seated during the bath.

TECHNICAL CHARACTERISTICS

- Non-slip surface and hole designed not to accumulate water
- Material: Aluminium frame, PVC seat.
- Adjustable seat height: 40-50 cm
- Total Height: 80-90 cm
- Length: 50 cm
- Depth: 44 cm
- Seat size: 50 x 28 cm
- Item Net weight: 2.55 Kg
- Max. weight capacity: 110 Kg

CORRECT USE

- The rubber ferrules were not designed to be needlessly and repeatedly removed. This could lead to its weakening and breakage.
- Shower chairs should always be used for their intended purpose. They must not be used for any other purpose. For example, they should not be used as a hanger to leave items or clothes, to rest the feet, etc.
- Do not make any modifications (mechanical alterations) to this item so as not to compromise its safety of use.
- Never exceed the maximum weight capacity.

PRECAUTIONS

- Before each use, you must make sure that the product is in perfect condition.
- Before each use, make sure the ferrules are in perfect condition. If they show wear, contact your distributor immediately for replacement or repair.
- Do not use the product in case of visible defect. In the event of missing, ill-fitting or damaged parts, seek advice from the establishment or distributor where you purchased the product.
- Check regularly that the screws are tight and secure.
- You must always sit straight and upright to avoid the danger of tipping or slipping.

ASSEMBLY

1. Place the seat top down on the floor.
2. Fix the legs with the screws.
3. Screw the backrest and the tubes
4. In order to place the backrest, screw the backrest tubes under the seat

CONTRAINDICATIONS

Not recommended when the user suffers from severe balance and stability problems.

CARE & MAINTENANCE RECOMMENDATIONS

- Clean and dry periodically. Remove dirt with a soft, damp cloth and neutral soap. Dry later. Do not use aggressive or abrasive chemical cleaners; they can damage the material of the product. Also, do not use greasy products that can degrade materials or make it slippery.
- When not in use, store the product in a dry place at room temperature.

LIFE SPAN

The expected life span of this product is 2 years under normal conditions of use, safety and maintenance. Beyond this period, the product can be used as long as it is in perfect conditions of use and maintenance. In any case, the product may require the replacement of certain parts or components due to normal wear in a period shorter than indicated. The useful life is influenced by its conditions of use and storage and by the regularity of its maintenance.

WARRANTY

The warranty is for a period of three years (6 months on batteries) from the date of the sales invoice, as long as the product has not been altered in its original configuration. Within the warranty period we will remedy any manufacturing defect free of charge, either by repairing, replacing parts or providing a new product, at our discretion. To opt for repair or replacement, the owner of the product must go to the retail establishment, which will be the only one in charge of managing the warranty with the manufacturer.

This warranty will not be valid in cases of improper use or handling of the product, deterioration of the product due to external agents such as water or other harmful chemicals, obstructive or corrosive products, due to inadequate maintenance, lack of cleaning, component loss or the use of non-original spare parts of our brand. For more information check our general conditions of sale.

WARNING

- If you observe any deficiency, anomaly or failure, immediately notify the vending establishment or distributor to proceed with its change or repair.
- Any serious incident related to the product must be reported to Sucesores de Manuel García, S.L. and to the competent authority of the Member State in which the user and/or the patient are established.

| | | | |
|---|-------------------|---|----------------------|
|  | Product Reference |  | Manufacturing date |
|  | Medical Device |  | Lot Number |
|  | EC Certificate |  | Instructions for use |
|  | Manufacturer |  | Precaution |
|  | Distributor |  | Importer |

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